

WHAT ARE THE BENEFITS OF NATURAL WELLNESS CUPS?



They are made of 100% safe medical silicone, that is not too soft for better disclosure inside, and not too hard, so they are absolutely not felt when used. Furthermore silicone supports the natural hydration of the vagina, as well as prevents the air penetration to the secretion, significantly reducing the risks of SSH.

The safety of the material of Natural Wellness cups for the skin and mucosa is confirmed by REACH certification.

HOW TO USE NATURAL WELLNESS?



(The cup can be boiled for best disinfection) Wash your hands

before first use.


before inserting or removing the bowl.



insert it into the vagina. Inside, should open up.



12 hours or as needed: slightly squeeze the bottom of the cup to break the vacuum and slowly pull it out.

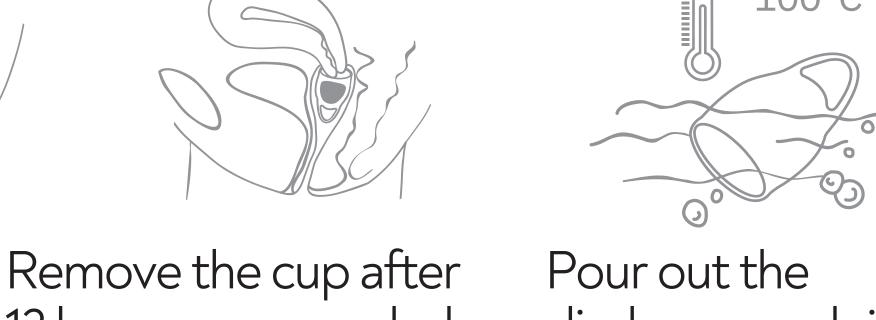

discharge and rinse the cup. At the end of menstruation, rinse the cup thoroughly and put it in a storage case. (You can boil the cup for 2-5 minutes for better sterilization) The Natural Wellness menstrual cups are suitable for women who lead an

active lifestyle: they are completely invisible under a swimsuit and protect against leaks even during the most active sports.



The sets include 2 cups with a volume of 15 ml and 20 ml. They are packed in cases that can be used to store cups. The case does not get wet, retains its shape and does not take up much space in a backpack or bag.

any unnecessary materials.

Case: - Moisture-proof

- Easily recycled

Also in the collection there are cups of 15 ml and 20 ml that are packed in individual cardboard boxes

without the use of plastic.

- Dries quickly

- Retains its shape

